

### CYDAL+ DISINFECTANT SPRAY (CY 2009)

**ACTIVE INGREDIENTS:** Isopropyl Alcohol  
**CONCENTRATION:** 70% v/v  
**WEIGHT:** 500ml

#### INTENDED USE

To clean and disinfect surfaces of non-invasive medical devices within healthcare and environment.

#### PRIMARY PACKAGING

· 24 bottles/carton

#### INSTRUCTIONS OF USING:

- Spray this product 6-8 inches from surface until thoroughly wet.
- Treated surfaces must remain wet for appropriate contact time listed below.
- Wipe or allow to air dry.
- No rinsing required.

#### STORAGE CONDITIONS:

- Store this product in a cool, dry area.
- Do not reuse or refill this container except as described in the directions for use.
- Offer for recycling or place in trash.
- Physical/Chemical Hazards Combustible.
- Do not use or store near heat or open flame.

**MDA REGISTRATION NO.:** GB6832522-99209



Test Protocol	Test Organism	Contact Time	Log % Reduction
EN1276	Staphylococcus aureus	30 Sec	99.999
	Pseudomonas aeruginosa	30 Sec	99.999
	Enterococcus hirae	30 Sec	99.999
	Escherichia coli	30 Sec	99.999
	Staphylococcus aureus (Methicillin resistant), MRSA	30 Sec	99.999
EN1650	Candida albicans	30 Sec	99.99

**NUR JAYA MEDIC SDN. BHD. (856525-A)**

SALES AND MARKETING OFFICE:  
No. 39, Jalan PJU 1A/16, Taman Perindustrian Jaya,  
47200 Petaling Jaya, Selangor Darul Ehsan, Malaysia.  
Tel: +603-7842 8641 | Fax: +603-7842 8642

*Quality Guaranteed. Create, Enhance & Sustain.*

MANUFACTURING SITE (PLANT):  
No. 2 & 2A, Lorong Industri Ringan Permatang Tinggi 12,  
Taman Industri Ringan Permatang Tinggi,  
14100 Simpang Ampat, Pulau Pinang, Malaysia.  
Tel: +604-502 2474

✉ sales@nurjayamedic.com  
🌐 www.nurjayamedic.com